SYMBOLS GLOSSARY:

Consult instructions for use

Do not use if package is damaged

Keep away from direct sunlight

Sterile barrier
Sterile Versiegelung
Barrière stérile
Steriele barrière
Barriera sterile
Barrera estéril


Medical device
Medizinprodukt
Dispositif médical
Medisch hulpmiddel
Dispositivo medico
Producto sanitario

Manufacturing date

Use by date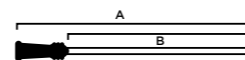

A	20cm/8"	40cm/16"
B	18cm/7"	35cm/14"

Manufacturer

Rx Only (US)
For prescription use only

US/UK/CA/AU

PRODUCT DESCRIPTION

Single use, hydrophilic urinary catheter with salt solution for activation.

MATERIAL

Hydrophilic coated urinary catheter. Polyolefinbased elastomer (POBE) with PVP coating. Sachet with NaCl and water.

INDICATIONS FOR USE

For short and long term bladder management with intermittent urinary catheterization.

PATIENT POPULATION

For adults, adolescents and children.

WARNINGS AND PRECAUTIONS

For single use only.

Once used the surface coating will deteriorate and is no longer sterile. Reuse may lead to discomfort, urethral damage or infection.

Do not use a product if the sterile packaging is broken or damaged. Urinary catheterization therapy is associated with an increased risk of urethral bleeding, trauma and/or infection.

LoFric catheters are for prescription use only.

Follow instructions and advice from your healthcare professional. Contact your prescriber if you experience difficulties.

ADVERSE REACTIONS

Common adverse reactions (>1/100) related to catheterization therapy includes urethral damage and urinary tract infection.

If unexpected discomfort, sign of trauma or infection occurs, discontinue use and consult your prescriber.

Any serious adverse reaction occurring when using the catheter should be reported to the manufacturer and your local health authority.

PACKAGE AND STORAGE

Store in their package in a dry place, at room temperature.

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5501009-A-2021-02-17

Find out more about our new, greener process at
www.wellspect.co.uk/reducedfootprint
www.wellspect.us/reducedfootprint

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UK: For more information: www.wellspect.co.uk, or call us at (0)1453 791763

US: For more information please visit www.wellspect.us, or call us toll free at 877-4-LOFRIC (456-3742).

AU: For more information please go to www.wellspect.com.au
Australian sponsor:
Dentsply Sirona Pty Ltd,
11-21 Gilby Road, Mount Waverley VIC 3149

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


OTHER MARKETS: For further information visit www.wellspect.com

Wellspect HealthCare
Aminogatan 1, P.O. Box 14,
SE-431 21 Mölndal, Sweden.
Tel: +46 31 376 40 00.
www.wellspect.com


Wellspect®
A REAL DIFFERENCE

MIX
Paper from responsible sources
FSC® C012075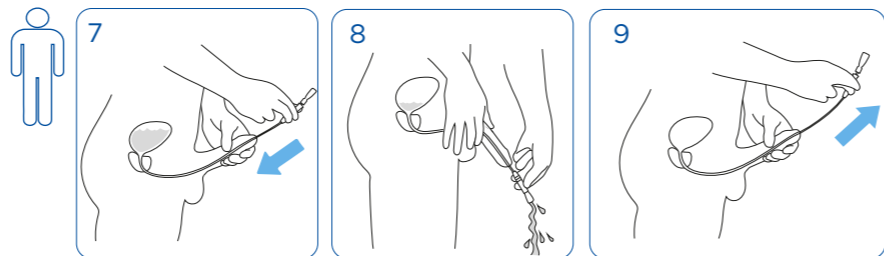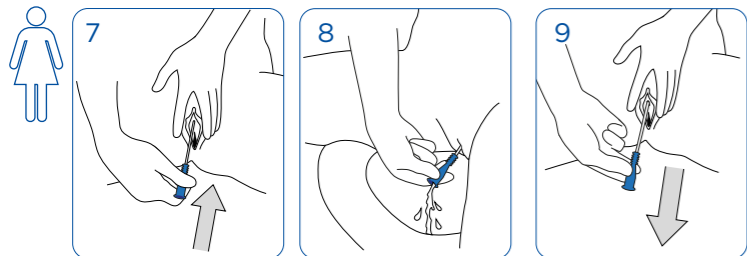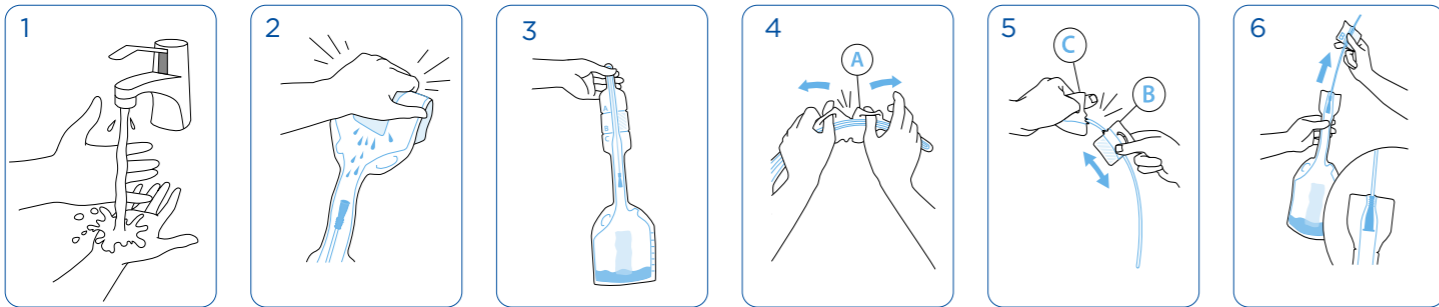

Tiemann/Coudé

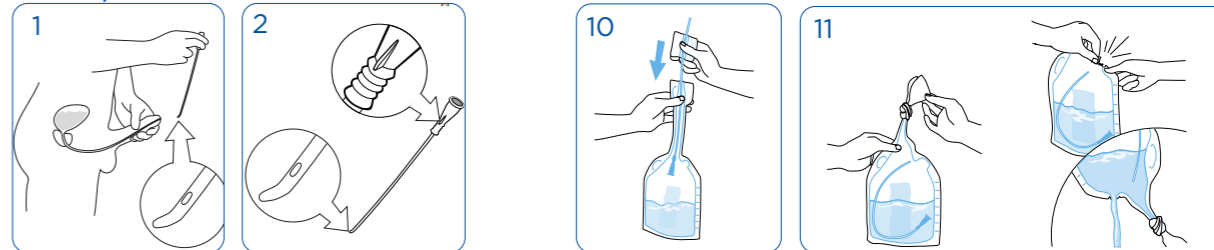US/UK/CA/AU INSTRUCTIONS FOR USE

1. Wash your hands thoroughly with soap and water.
2. Hold the product upright. Fold the sachet and squeeze. Let the salt solution run down to the catheter.
3. Turn the product upside down to allow the salt solution to drain into the collection bag.
4. Use loops at indentation "A" to open and uncover the tip of the catheter.
5. Use loops at indentation "B/C" to open. Use section "B" as insertion grip.
6. Gently pull the catheter out of the package until the funnel comes to a stop, to seal between catheter and collection bag.

FEMALE

7. Spread the labia and locate the urethra just above the vaginal opening. With the other hand, insert the catheter slowly into the urethra.
8. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
9. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.
10. Push the catheter back into the collection bag.
11. Before disposal, empty the bag through the neck, or tie a knot to seal the bag and tear at the indentation. Dispose appropriately (local regulations may vary).

MALE

7. Lift the penis towards the stomach to straighten the urethra. Slowly insert the catheter into the urethra. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
8. Angle the penis down as urine begins to flow through the catheter.
9. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.
10. Push the catheter back into the collection bag.
11. Before disposal, empty the bag through the neck, or tie a knot to seal the bag and tear at the indentation. Dispose appropriately (local regulations may vary).

Special instruction for Tiemann/Coudé catheter, with slightly curved tip.

A special technique is required when using a Tiemann/Coudé. Speak to your healthcare professional for training and advice.

1. Note where the marker on the funnel is in relation to the curved catheter tip before inserting the catheter. It will guide you keeping the curved tip in the right direction during use.
2. Keep the curved tip upwards towards the stomach during insertion and throughout catheterization, including withdrawal. Or follow specific instructions given by your healthcare professional.

IMPORTANT INFORMATION

Use only the salt solution included to wet the catheter. Do not place the catheter on paper or cloth as these materials can absorb the wet coating.

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